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The Prodox **CPV, CCoV, Gia Ag** test is used to detect parvovirus, coronavirus and G. intestinalis antigens in dogs feces. Antigen (Ag) is a foreign substance that stimulates the body to produce antibodies.

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Storage of Tests:

Store the test kits in a temperature 2-30°C. Do not freeze, protect from direct sunlight.

Sample Storage:

The collected sample should be tested immediately.

USE CAUTION:

Samples are considered infectious material! Expiration date is indicated on the package. Tests are for animal use only.

Performance:

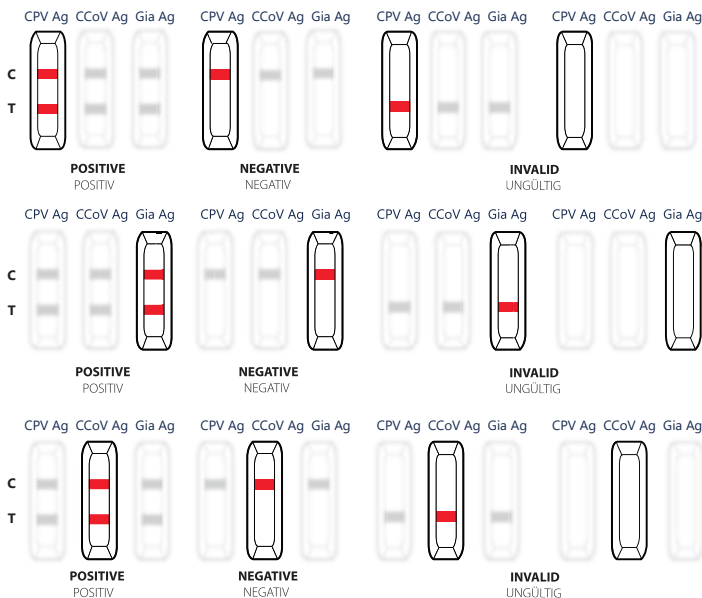
Test **specificity** is the ratio of the true negatives to the sum of the true negatives and false positives combined and was: for CPV 100%, CCoV 99.2% & Giardia 100% for this assay. It means that 100%, 99.2% & 100% of samples from healthy individuals with this test are marked as healthy / negative. The **sensitivity** of the test is the ratio of the true positives to the sum of the true positives and the false negatives. The sensitivity: of CPV 97.6%, CCoV 94.9% & Giardia 95.7%, estimated in this test, means that such a percentage of sick individuals will be positively diagnosed.

LIMITATIONS:

1. The test result should not be the sole source of diagnosis, as with other diagnostic tests, the result should be interpreted by the veterinarian with the other clinical data.
2. False positive result may occur when:
 - there is a cross-reactivity,
 - antigens of another pathogens are present.
3. False negative result may occur when:
 - the specimen has been sampled incorrectly,
 - antigens levels in the sample are too low,
 - the specimen has been sampled too early or too late, the tested antigen is not present in the feces,
 - GIARDIA: the sample has been taken too soon and in the tested feces the g.intestinalis antigen is not yet present.

Due to the possibility of false-positive or false-negative results, cassette test results may be confirmed by other methods.
5. GIARDIA: A positive result in the cassette test may appear up to 8-12 weeks after infection, so treatment of the infection should be controlled by other diagnostic methods.
6. Invalid test result may occur when:
 - the specimen was sampled incorrectly,
 - sediment has been placed in the test well "S",
 - not enough drops of sample material have been placed in the test well "S".
7. If the symptoms persist despite the negative or invalid results, it is recommended to do the test again within 24-48h and contacting your veterinarian.
8. GIARDIA: In some cases, positive test result may only indicate that the pet had contact with microorganisms and doesn't have to be equivalent of the disease development.
9. Negative result does not exclude a possibility of infection.
10. Positive result does not exclude a possibility of coexisting infection with different pathogens.
11. Each positive result should be consulted with a veterinarian.

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EN

Positive result: there will be 2 lines, 1 in control field (C) and 1 in the test field (T).
Look closely! The presence of any line in test field (T), even very vague, means that the result is considered positive.
Negative result: only 1 clear line in control field (C), no line in test field (T).
Invalid result: no color line in control field (C), the invalid result is not affected by the appearance of the test field (T) (presence or absence of a line in this field).

PRODOX Professional Vital PetCare

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EN

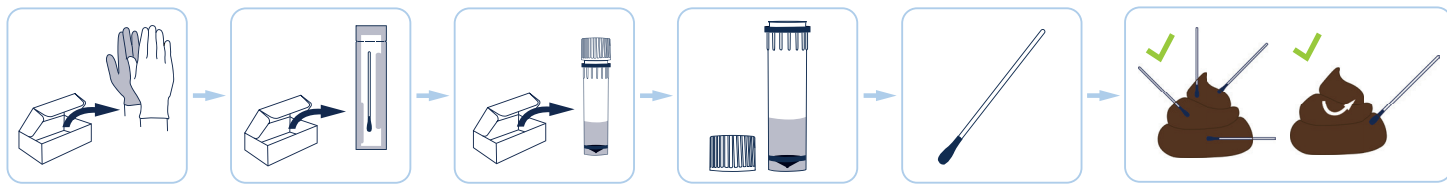
Test principle:

The test is performed by a lateral flow immunochromatic method (to detect the presence or absence of the target Ag in the sample). The test cassette has a window (S) into which the sample is dropped and two fields: test field (T) and control field (C). When the sample is placed in the (S) window, the liquid will flow laterally into test field (T). If the antigen is present in the sample, a visible (T) line will appear.

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EN

Take out the gloves from the box and put them on.

Take out the swab from the box.

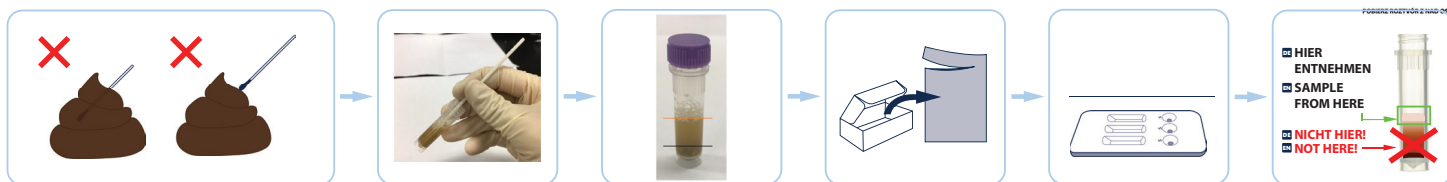
Take out the tube with the buffer solution from the box.

Twist off the tube cap.

Take the feces sample properly with the swab.

CORRECT:

To correctly prepare the feces sample you need to take the sample from many spots, rubbing the swab on the outer layer of the sample.

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EN

INCORRECT:

Don't take the sample from one side only. Don't insert the swab too deeply into the feces.

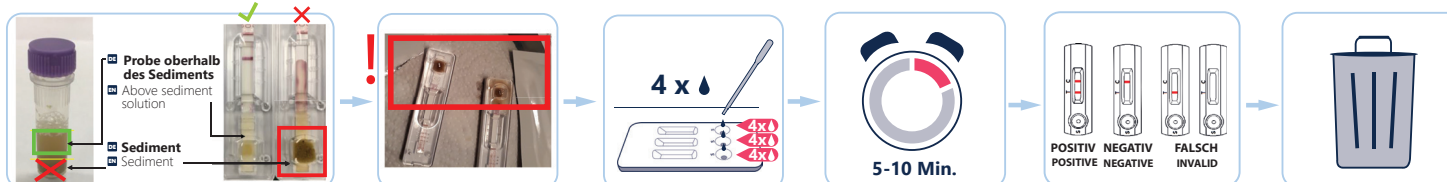
Twist off the assay tube cap. Insert the swab into the tube with the buffer solution and mix the sample well.

Leave the tube for one minute to let any sediment settle on the bottom of the container.

Open the box and take out the pouch containing the test.

Take out the test cassette and place it on a flat and clean surface.

Take the tube with the sample solution. **Using the pipette from the test pouch, take the sample from above the sedimentation.**

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EN

Incorrect, sediment blocks the flow.

Undissolved sediment blocks the flow into the test window. **Take a sample as instructed using the solution from above the sediment.**

Add 4 drops into each test window (S).

Read the results after 5-10 minutes. **WARNING! Results read after 10 minutes may be invalid!**

Positive result: 2 lines, 1 in field (C) and 1 in the field (T).
Negative result: only 1 clear line in field (C), no line in field (T).
Invalid result: no color line in field (C).

After use collect all the test components in an empty trash bag, close it precisely. Dispose it in residual waste.

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PRODOX
Professional Vital PetCare

Prodox Vital GmbH
Eichweberstr. 4
D-25821 Bredstedt

Email: info@prodoxvital.de

www.prodoxvital.de