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The Prodox **CPV Ag** test is used to detect parvovirus antigen in dogs feces. Antigen (Ag) is a foreign substance that stimulates the body to produce antibodies.

RECOMMENDATIONS AND PRECAUTIONS:

1. Before using the product, read the instruction manual. Follow the instructions carefully. Otherwise, the results may be inaccurate.
2. Do not use the test kit beyond its expiration date.
3. Ensure all items in the test kit and specimens are at the same temperature before testing - this will take approximately 15-25 minutes.
4. The test should remain in the sealed pouch until use. Perform the test immediately after removing the test cassette from the pouch. Touch only the sides of the test cassette.
5. Do not use the test kit if the pouch is open or damaged.
6. Do not mix test kit components from different lot numbers.
7. This test kit is for external use only. Do not swallow any of the test kit components.
8. Avoid getting buffer solution in the eyes or on the skin. In case of accidental contact, rinse with plenty of water to avoid skin irritation. In case of eye contact, flush eyes with plenty of water for at least 15 minutes. If swallowed, drink plenty of water. Seek medical attention.
9. In the event of buffer solution, clean up spills thoroughly with a disinfectant.
10. Use only the buffer solution (from the tube provided in the kit) for testing. The use of other liquids (e.g., water) may yield inaccurate results.
11. Keep the test kit out of the reach of children.
12. The test kit is for single use only. Do not reuse any of the components of the test kit.
13. Dispose of the used test kit components in a sealed bag for mixed waste.
14. Wash your hands thoroughly before and after testing.

Storage of Tests:

Store the test kits in a temperature 2-30°C. Do not freeze, protect from direct sunlight.

Sample Storage:

The collected sample should be tested immediately.

USE CAUTION:

Samples are considered infectious material! Expiration date is indicated on the package. Tests are for animal use only.

Performance:

Test specificity is the ratio of the true negatives to the sum of the true negatives and false positives combined and was: for CPV 100% for this assay. It means that 100% of samples from healthy individuals with this test are marked as healthy / negative. The **sensitivity** of the test is the ratio of the true positives to the sum of the true positives and the false negatives. The sensitivity: of CPV 97.6%, estimated in this test, means that such a percentage of sick individuals will be positively diagnosed.

LIMITATIONS:

1. The test result should not be the sole source of diagnosis, as with other diagnostic tests, the result should be interpreted by the veterinarian with the other clinical data.
2. False positive result may occur when:
 - there is a cross-reactivity,
 - antigens of another pathogens are present.
3. False negative result may occur when:
 - the specimen has been sampled incorrectly,
 - antigens levels in the sample are too low,
 - the sample has been taken too soon and in the tested feces the antigen is not yet present.
4. Due to the possibility of false-positive or false-negative results, cassette test results may be confirmed by other methods.
5. Invalid test result may occur when:
 - the specimen was sampled incorrectly,
 - sediment has been placed in the test well "S".
 - not enough drops of sample material have been placed in the test well "S".
6. If the symptoms persist despite the negative or invalid results, it is recommended to do the test again within 24-48h and contacting your veterinarian.
7. Negative result does not exclude a possibility of infection.
8. Positive result does not exclude a possibility of coexisting infection with different pathogens.
9. Each positive result should be consulted with a veterinarian.

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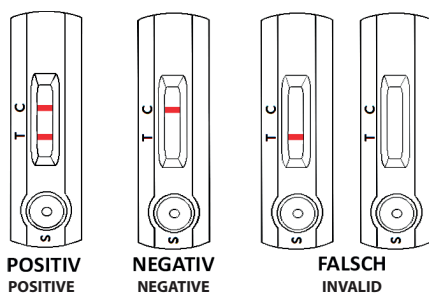
INTERPRETATION OF RESULTS

EN Positive result: there will be 2 lines, 1 in control field (C) and 1 in the test field (T). **Look closely! The presence of any line in test field (T), even very vague, means that the result is considered positive.**

Negative result: only 1 clear line in control field (C), no line in test field (T).

Invalid result: no color line in control field (C), the invalid result is not affected by the appearance of the test field (T) (presence or absence of a line in this field).

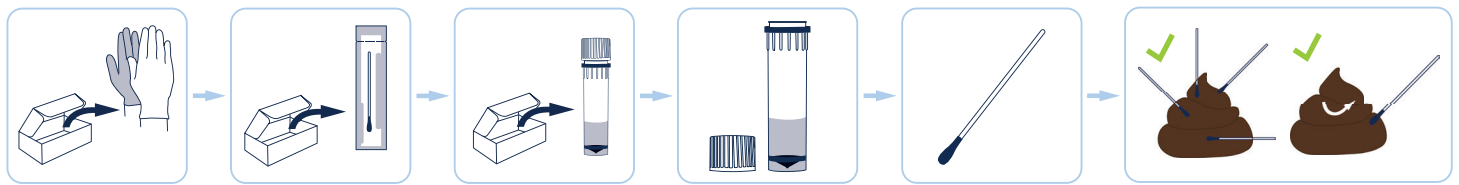
EN Test principle:

The test is performed by a lateral flow immunochromatic method (to detect the presence or absence of the target Ag in the sample). The test cassette has a window (S) into which the sample is dropped and two fields: test field (T) and control field (C). When the sample is placed in the (S) window, the liquid will flow laterally into test field (T). If the antigen is present in the sample, a visible (T) line will appear.

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
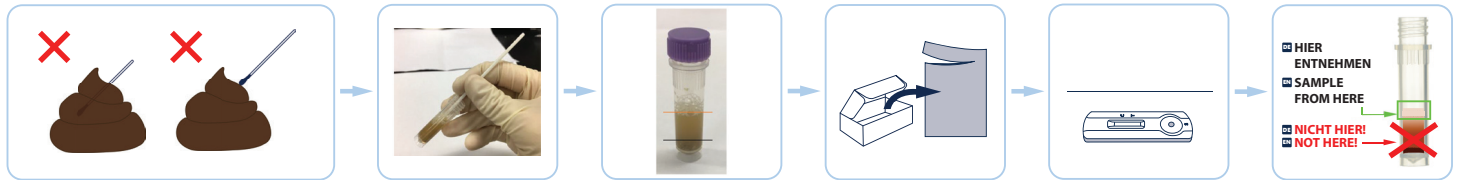
EN Take out the gloves from the box and put them on. Take out the swab from the box. Take out the tube with the buffer solution from the box. Twist off the tube cap. Take the feces sample properly with the swab. **CORRECT:** To correctly prepare the feces sample you need to take the sample from many spots, rubbing the swab on the outer layer of the sample.

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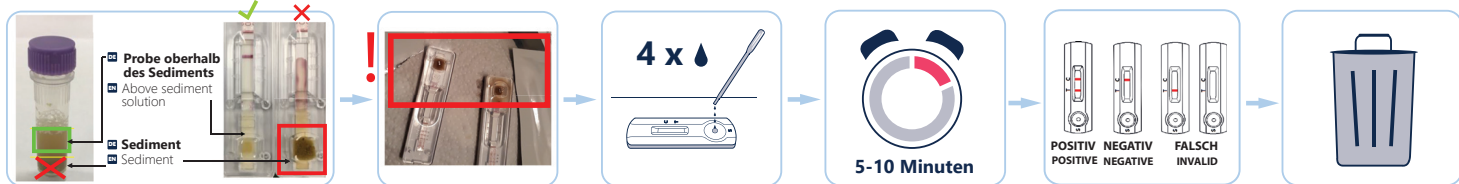EN INCORRECT: Don't take the sample from one side only. Don't insert the swab too deeply into the feces. Twist off the assay tube cap. Insert the swab into the tube with the buffer solution and mix the sample well. Leave the tube for one minute to let any sedimentation settle on the bottom of the container. Open the box and take out the pouch containing the test. Take out the test cassette and place it on a flat and clean surface. Take the tube with the sample solution. **Using the pipette from the test pouch, take the sample from above the sedimentation.**

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EN Incorrect, sediment blocks the flow. Undissolved sediment blocks the flow into the test window. **Take a sample as instructed using the solution from above the sediment.** Add 4 drops into the test window (S). Read the results after 5-10 minutes. **WARNING! Results read after 10 minutes may be invalid!** Positive result: 2 lines, 1 in field (C) and 1 in the field (T). Negative result: only 1 clear line in field (C), no line in field (T). Invalid result: no color line in field (C). After use collect all the test components in an empty trash bag, close it precisely. Dispose it in residual waste.

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PRODOX
Professional Vital PetCare

Prodox Vital GmbH
Eichweberstr. 4
D-25821 Bredstedt

Email: info@prodoxvital.de

www.prodoxvital.de